

[REDACTED]

[REDACTED]

[REDACTED]

Date : Thursday, August 18, 2016

Attention [REDACTED]

Job ref: CE57936
Repairs to BM900 Haag Streit Slit Lamp S/N 01093

Following the recent repairs to your instrument, I attach below, a repair description and general condition report.

An investigation into the burning smell from the power supply revealed that the mains power switch contacts had failed, causing it to melt and creating the noted smell. (see attached Photos)


Please note that although this is age and use related wear, I have opted to replace the switch with a much heavier and more robust type.

I have also performed a general inspection of the instrument and made some minor preventative corrections.



Note the switch pictured above is black and burnt



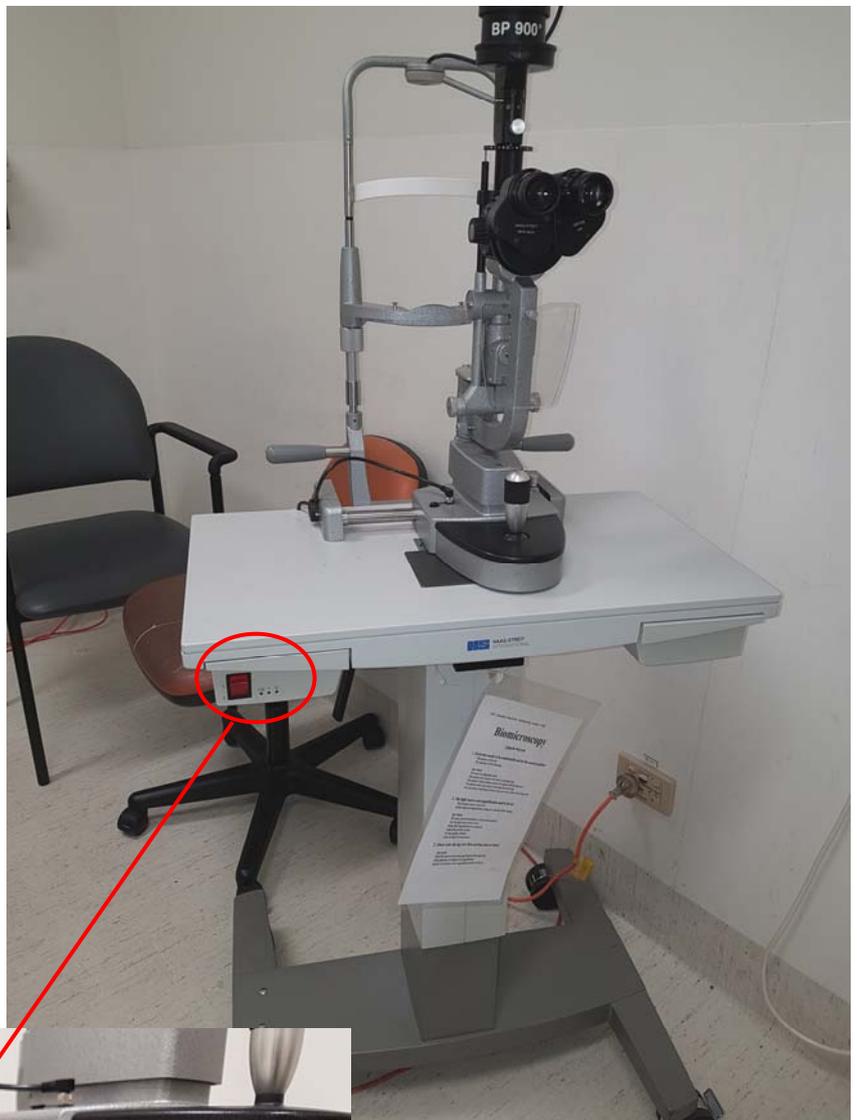

Note new larger switch pictured below.



Please feel free to contact me with any further questions.

Kind regards
Mario Sotiriadis



[Redacted]		[Redacted]		Service No.		S0160816	
Contact		[Redacted]				Date	16/08/2016
Customer Name		[Redacted]					
Address		[Redacted]					
City	[Redacted]		State	VIC		Postcode	[Redacted]
Phone	[Redacted]			Fax	[Redacted]		
Email		[Redacted]					

Slit Lamp

Make	Haag Streit		Model	BP 900		Serial #	01093
------	-------------	--	-------	--------	--	----------	-------

Service Check List	✓	Comments
Tracks & Wheels	✓	OK
Glide Plate	✓	OK
Swing Arm Movement	✓	OK
Lamp tower	✓	OK
Slit, mask & filters	✓	OK
Electrical contacts & Lamp	X	Burnt and poor electrical contacts required cleaning
Drive Voltage	✓	OK
Cabling	✓	OK
Power Supply	X	Faulty Mains Power Switch (Removed repaired offsite and returned)
Optics, Convergence	✓	OK
Focal & Light position	✓	OK
Brightness Control	✓	OK
Chin Rest	✓	OK
Fixation Light	✓	OK
Joystick	✓	OK
Height adjustment	✓	OK
Eye Pieces	✓	OK
PD & Zoom Adjustment	✓	OK
Clean and Lubricate All Movements		
Clean Optics		
Teaching Tube	✓	Not Fitted
Beam Splitter	✓	OK

◀ General Condition of Instrument ▶

Poor	1	2	3	4	5	6	7	8	9	10	Excellent
Date		Start			On Site			Off Site		Finished	
16/08/16		10:15:00			10:25:00			11:00:00		11:25:00	
16/08/16					12:10:00			12:40:00			
16/08/16		14:10:00			14:20:00			14:40:00		15:00:00	

Parts Required	Part Number	Qty
Illuminated mains rocker switch 240V15A DPDT		1